



S-200	ZEUS-400	ZEUS-300	ZEUS-200
/ 300Ω	400W / 300Ω	300W / 300Ω	200W / 300Ω
/ 300Ω	250W / 300Ω	230W / 300Ω	150W / 300Ω
/ 300Ω	200W / 300Ω	180W / 300Ω	120W / 300Ω
/ 300Ω	150W / 300Ω	120W / 300Ω	100W / 300Ω
/ 300Ω	120W / 300Ω	100W / 300Ω	100W / 300Ω
/ 300Ω	100W / 300Ω	80W / 300Ω	80W / 300Ω
/ 100Ω	99W / 100Ω	80W / 100Ω	80W / 100Ω
/ 200Ω			



※ In case of Zeus-200, receptacle part is little different than Zeus-400/300, When customer buys Zeus-200, please check.

The ZEUS series provide safety flexibility and convenience by powerful and stable technology.

- Main Frequency : 400, 500KHz
- Power Source : AC120V or AC230V, 50Hz or 60Hz
- Protection class : Class 1, Type CF
- Dimension(WxDxH mm) 330x455x150
- Weight : 13.0Kg



Application

- General Surgery
- Dermatology
- Vascular Surgery
- Gynecology
- ORL/ENT
- Pediatric Surgery
- Urology
- Orthopedics
- Oral Surgery
- Plastic Surgery
- Endoscopy

Safety Function

- A fuse which is built in a power circuit board prevents over current from flowing through the equipment
- When the plate attached to the patient is disconnected from the unit, red alarm light begins flickering. If press any button of the twin button handle or press any pedal of double foot switch, alarm sound is made but the unit doesn't work.
- REM(Return Electrode Monitoring) function monitors the size of contacting area between a patient and the pad. If size is inappropriate, it automatically blocks the high-frequency current to minimize the danger of burning incident.
- To protect a patient, the case is fully grounded so that a leakage current can flow into the earth.